

CHLOR-X

2% CHLORHEXIDINE SOLUTION

PREVESTDenPro®
THE FUTURE OF DENTISTRY

Product Description : Chlor-X is a chlorhexidine digluconate based water - alcoholic solution for irrigation and cleaning of root canals.

Composition : Chlorhexidine Digluconate, Ethyl Alcohol and Water

Indications :

Irrigation, cleaning of root canals.

Contraindications :

Hypersensitivity to chlorhexidine digluconate.

Instructions for use :

After root canal preparation, Chlor-X should be delivered inside the canal using a safe-ended endodontic syringe. Plug with a cotton pellet and fill with a temporary cement Caution should be taken to avoid interaction of Chlor-X and NaOCl in root canals which will create the formulation of Para chloro aniline that bears toxic and carcinogenic effect.

Storage Conditions :

Keep bottle tightly closed

Keep away from direct heat & light

Keep in a cool place between 10°C to 24°C .

Shelf Life :

3 years from the date of manufacturing.

Presentation :

Ref: 40026

1 x 100ml Bottle, Dispensing Dropper.

Ref: 40026-1

1 x 500ml Bottle.




Disposal conditions :



To be disposed off based on the prevailing rules of the country in which it is used.


 **Prevest DenPro Limited**
Unit I, 3B, Industrial Estate,
Digiana, Jammu-180010, India
Unit II, EPIP, Bari Brahmna,
Jammu-181133, India
www.prevestdenpro.com
Customer Care No. : +91 1923 222779 (10AM-5PM)
Email : info@prevestdenpro.com



EC REP
Obelis s.a
Boulevard General Wahls 53
1030 Brussels, Belgium
Tel: +(32) 2 732 59 54, Fax: +(32) 2 732 60 03
E-mail : mail@obelis.net



Revision :	03
Date :	27-02-2017

LOT Symbol for "BATCH CODE"  Symbol for "USE BY"  Symbol for "NON STERILE"  Symbol for "KEEP DRY"

 Symbol for "CAUTION"  Symbol for "CONSULT INSTRUCTION FOR USE" **REF** Symbol for "CATALOGUE NUMBER"

EC/REP SYMBOL FOR EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE  Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"

 Symbol for "KEEP OUT OF THE REACH OF CHILDREN"  Symbol for "MANUFACTURER NAME & ADDRESS"

 Symbol for "TEMPERATURE LIMITATION"  Symbol for "KEEP AWAY FROM SUNLIGHT"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.