

HIFLEX TRACING STICKS

Functional and Relining
Impression Material

PREVESTDenPro®
THE FUTURE OF DENTISTRY

Product Description : Because of its unique slow setting characteristics HIFLEX GREEN STICKS / Border moulding compound enables of extremely accurate impressions. This makes it particularly useful as an impression material for rebasing and muscle trimming. May also be used for general impression such as border moulding in dentures, rebasing procedures, for recording full depth of sulcus and posts.

Composition : Thermoplastic Resin, Stearic Acid, Talcum, Colorant and excipients.

Instructions for Use :

For border extensions on impression trays :

1. Remove the flange and tissue contact surface from the denture to a depth of 2-3 mm using steel burs or abrasive points.
2. Soften the material over soft flame and cover the relined surface of the denture with HIFLEX GREEN STICKS.
3. Place the softened, material on the impression tray and again soften it by passing through hot water at (45 °C / 113 °F). Insert the prepared tray in mouth with firm and even pressure, hold for 2 min.
4. Instruct the patient to perform muscle trimming movements.
5. Trim the excess material & apply adhesive to prepare tray for final impression.

For Reabsing Procedure : The denture should be retained in the mouth for 1 day to allow a truly functional impression to be formed under normal chewing conditions.

Note : Advice patient to avoid hot drinks during this period.

Contraindications : In rare cases the product may cause sensitivity to some patients. If such reactions are experienced, discontinue the use of the product and refer to a physician.

Notes :

1. HIFLEX GREEN STICKS has a fast colour, no taste, no smell and is non-irritating to the patients's mouth.
2. To avoid green sticks becoming too sticky do not over heat.
3. If HIFLEX GREEN STICKS adhere to the denture, let it set hard and remove with laboratory bur and hand instruments. Remove residual traces using solvents.

Storage :

Store at temperature between 10°C to 24°C. Close the cap carefully after use. Keep away from moisture. Shelf life not affected after opening if stored in a cool dry place.

Shelf life : 3 years from the date of manufacturing.

Presentation :

Ref. 70011














10 sticks, Net weight 90g (each stick 9 g)

Caution : In case of contact with eyes, remove and flush with water.

 **Prevest DenPro Limited**
Unit II, EPIP, Bari Brahmna,
Jammu-181133, India.
www.prevestdenpro.com
Customer Care No. : 1800-180-7172
Email : customercare@prevestdenpro.com

CE

Revision :	03
Date :	23-05-2019

-  Symbol for "BATCH CODE"  Symbol for "USE BY"  Symbol for "NON STERILE"  Symbol for "KEEP DRY"
-  Symbol for "CAUTION"  Symbol for "CONSULT INSTRUCTION FOR USE"  Symbol for "CATALOGUE NUMBER"
-  SYMBOL FOR EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE  Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"
-  Symbol for "KEEP OUT OF THE REACH OF CHILDREN"  Symbol for "MANUFACTURER NAME & ADDRESS"
-  Symbol for "TEMPERATURE LIMITATION"  Symbol for "KEEP AWAY FROM SUNLIGHT"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.