

ZITEMP

ZOE CEMENT QUICK SET

PREVESTDenPro®
THE FUTURE OF DENTISTRY

Product Description : Zitemp is quick set easy mix zinc oxide-eugenol composition for thermal insulation and temporary restorations.

Composition :

Powder is Zinc Oxide and liquid contains Eugenol and Acetic Acid

Indications :

Temporary restoration, temporary cementation of crowns, cavity liner under most restorative materials.

Instructions for Use :

Mix Powder and liquid in a ratio of 0.32 to 0.08 (1scoop powder and 3-4 drop liquid) for about 45- 60 seconds to obtains putty consistency. Place the putty into the cavity and remove excess of the product immediately. Allow hardening for 11-12 minutes. Trim the restoration with a tungsten carbide bur.

Warning :

Avoid prolonged or repeated contact with skin (allergic contact dermatitis), and eyes. Avoid prolonged inhalation. Do not take internally. Contact with the skin may cause irritation, burns or hypersensitivity. If contact with skin occurs, immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs, discontinue use and seek medical attention. If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician. It should not be used with patients who have a history of severe allergic reaction to any of the components.

Contraindications : It is contraindicated for use with patient who have a history of severe allergic reaction to any of the components.

Precautions :

Zitemp is a Eugenol based composition and therefore it should not be used as a liner under resin based filling materials. Avoid contact with eyes. Do not invert the bottle upside down while opening to avoid excess dispensing of the powder and avoid excessive force while dispensing the liquid from the dropper.

Storage Conditions : Store at temperature between 10°C to 24°C. Close the cap carefully after use. Keep away from moisture. Shelf life not affected after opening if stored in a cool dry place.

Shelf Life : 3years under proper storage.

Presentation:





Ref: 30016




15g Powder, 10ml Liquid, Spoon, Mixing Pad & Dispensing Dropper.



Disposal conditions : To be disposed off based on the prevailing rules of the country in which it is used.



Prevest DenPro Limited
Unit II, EPIP, Bari Brahmana,
Jammu-181133, India.
www.prevestdenpro.com
Customer Care No. : 1800-180-7172
Email : customercare@prevestdenpro.com



Revision :	07
Date :	23-05-2019

 Symbol for "BATCH CODE"  Symbol for "USE BY"  Symbol for "NON STERILE"  Symbol for "KEEP DRY"

 Symbol for "CAUTION"  Symbol for "CONSULT INSTRUCTION FOR USE"  Symbol for "CATALOGUE NUMBER"

 SYMBOL FOR EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE  Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"

 Symbol for "KEEP OUT OF THE REACH OF CHILDREN"  Symbol for "MANUFACTURER NAME & ADDRESS"

 Symbol for "TEMPERATURE LIMITATION"  Symbol for "KEEP AWAY FROM SUNLIGHT"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.