

Size : 105 x 265mm

CITROGEN GEL

Dentine conditioner and chelating agent
(20% Citric Acid Gel)



Product description :

Citrogen Gel is a 20 % Citric Acid gel used as a chelating agent for removal of smear layer and conditioning of dentine surfaces.

Composition : Citric acid, Thickening agent, Flavoring agents, Coloring agent and water.

Intended Purpose / Intended use: It is used as a chelating agent for removal of smear layer and conditioning of dentine surfaces.

Indications for use :

- Conditioning of prepared root canal systems
- Removes mineral and smear layers
- Lubrication during root canal preparation
- Conditioning of dentine surfaces

Instructions for use:

For Root canal procedures: Insert the product into the root canal using the syringe with an applicator tip while mechanically preparing the canal. Do not extrude the gel beyond the apex. Adequate care should be taken to restrict the material within the canal. After using the product, rinse the canal with physiological saline. Citrogen Gel is packaged in a multiple dose container (Syringe). Opening and closing of the container does not affect the shelf life of the device. The dispensed material shall not be placed back in container and the dose once applied and used shall not be reused.

Conditioning of dentine surfaces: Apply the Gel on the dentinal surfaces to the primed or conditioned . Brush it lightly with a Microbrush and leave it in place for a period of 3-5 minutes. Rinse the preparation with physiological saline.

Warnings :

1. The material may cause sensitization in susceptible people. Do not use Citrogen Gel in people allergic to components of this product.
2. Avoid eye contact to prevent irritation. In case of contact with eyes, rinse immediately with plenty of water and seek medical attention. Avoid skin contact to prevent possible allergic response. Material may cause irritation and therefore contact with skin and mucous membrane should be avoided .In case of contact, immediately remove material with cotton and wash thoroughly with water and soap. In case of skin sensitization or rash, discontinue the use and seek medical attention.
3. Do not swallow or take internally. If accidental swallowing occurs, drink lots of water. If nausea or illness develops, seek medical attention.

Precautions :

1. Always use a rubber dam for endodontic therapy. Prevent ingestion of Citric Acid gel.
2. Prevent getting gel in the eyes or leaving it on the soft tissues for an extended period of time.
3. Do not extend tip to pass the root apex or express citric acid gel pass the apex.
4. Do not follow citric acid rinse with Sodium Hypochlorite or vice versa.
5. The tips are disposable and intended for single use.
6. Use protective gloves, glasses and clothing.
7. Take special care when using this product to children because of the danger of excessive softening of tooth hard tissues. The product is fully biodegradable. Keep out of reach of children.

Contraindications : Do not use the product for patients with hypersensitivity to the product's components. Do not leave the product in the canal or the tooth cavity between visits. This may result in excessive softening of dentin. Restrict the use of the product to a maximum of 5 minutes, when used as a conditioning agent

Storage conditions : This product should be used at room temperature. Contact with water should be strictly avoided. Inadequate storage conditions could shorten shelf life and product performance. We recommend to store at temperatures between 10°C (50°F) and 24°C (75°F) and keep syringe cap tightly closed. .

Shelf Life : 2 years from the date of manufacturing.

Presentation:

Ref : 40045

1 x 10ml Syringe, 5 x Applicator Tips

Ref : 40045-1

1 x 30ml Syringe, 10 x Applicator Tips, 2 x 5ml Empty Syringes

Disposal conditions: To be disposed off based on the prevailing rules of the country in which it is used.

Prevest DenPro Limited
Unit II, Export Promotion Industrial Park,
(EPIP), Bari Brahmna, Jammu-181133,
India. www.prevestdenpro.com
Customer Care No. : + 91 8899074151
Email : customercare@prevestdenpro.com
Made in India

EC REP
Obells s.a
Boulevard General Wahis 53
1030 Brussels, Belgium
Tel: +(32) 2 732 59 54, Fax: +(32) 2 732 60 03
E-mail : mail@obells.net

CE
0123

Revision : 03
Date : 16-04-2024

UDI Symbol for "UNIQUE DEVICE IDENTIFIER"

- LOT** Symbol for "BATCH CODE" **USE-BY DATE** Symbol for "USE-BY DATE" **NON STERILE** Symbol for "NON STERILE" **KEEP DRY** Symbol for "KEEP DRY"
- CAUTION** Symbol for "CAUTION" **CONSULT INSTRUCTION FOR USE** Symbol for "CONSULT INSTRUCTION FOR USE" **REF** Symbol for "CATALOGUE NUMBER"
- EC REP** SYMBOL FOR "AUTHORIZED REPRESENTATIVE IN THE EUROPEAN COMMUNITY/EUROPEAN UNION" **DO NOT USE IF PACKAGE IS DAMAGE** Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"
- KEEP OUT OF THE REACH OF CHILDREN** Symbol for "KEEP OUT OF THE REACH OF CHILDREN" **MANUFACTURER** Symbol for "MANUFACTURER"
- TEMPERATURE LIMIT** Symbol for "TEMPERATURE LIMIT" **KEEP AWAY FROM SUNLIGHT** Symbol for "KEEP AWAY FROM SUNLIGHT" **MD** Symbol for "MEDICAL DEVICE"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding. For feedback, complaints and reporting of unexpected events contact customer care.