

ORADOX

Denture Cleansing Tabs

PREVESTDenPro®
THE FUTURE OF DENTISTRY

Product Description : Oradox denture cleansing tabs are specifically designed to clean dentures and removable dental appliances. These cleansing tabs help to remove stains, plaque and debris accumulated on dentures and dental appliances.

Composition : Sodium bicarbonate, Citric acid, Sodium percarbonate monohydrate, TAED, Sodium benzoate, Sodium lauryl sulphate, PVP, EDTA, Flavoring and Coloring agent

Intended Purpose / Intended Use : It is intended to be used as a daily cleanser for removable prosthetic dental appliances such as dentures & removable partial dentures.

Indications for Use : It is used to clean and remove food debris, plaque, stains and bad odour from full or partial dentures and oral prosthetics.

Instructions for Use : Place the denture in a glass of 150ml of warm water (33°C to 47°C) ensuring that it is fully immersed and immediately drop one oradox denture cleansing tablet in the water. Soak for a minimum of 15 minutes to ensure reduction of the biofilm. Acrylic appliances can be soaked overnight. Brush can be used if necessary. Remove the denture from the solution and rinse well under water before oral application.

Warnings : Harmful if swallowed.

Causes severe skin burns and eye damage.

IF SWALLOWED: Rinse mouth. Do NOT induce vomiting.

IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water/shower.

IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses if present and continue rinsing.

Get medical advice/attention if you feel unwell.

Precautions :

Discard solution after use.

Rinse denture well before replacing it in mouth

Do not place tabs or solution in mouth

Do not drink solution or use it as a mouthwash

Contraindications : Do not use on patients with known allergies to any of the ingredients.

Storage Conditions : Store at temperature between 10°C(50°F) to 24°C(75°F). Keep away from moisture and heat. The shelf life of the product is valid only if left unopened and correctly stored in a cool dry place.

Shelf Life : 2 years from the date of manufacturing.

Presentation :

Ref: 80016

1 x 15 Tabs in Desiccant Tube (Fresh Mint)

Ref. : 80017

1 x 30 Tabs in Desiccant Tube (Fresh Mint)

Ref. : 80096

1 x 10 Tabs in Desiccant Tube (Fresh Mint)

Disposal Conditions : To be disposed off based on the prevailing rules of the country in which it is used.

Prevest DenPro Limited

Unit II, Export Promotion Industrial Park,
(EPIP), Bari Brahmna, Jammu-181133,

India. www.prevestdenpro.com

Customer Care No. : + 91 8899074151








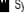


Email : customercare@prevestdenpro.com

Made in India

CE

Revision :	01
Date :	09-04-2024

UDI Symbol for "UNIQUE DEVICE IDENTIFIER"

- LOT** Symbol for "BATCH CODE"  Symbol for "USE-BY DATE"  Symbol for "NON STERILE"  Symbol for "KEEP DRY"
-  Symbol for "CAUTION"  Symbol for "CONSULT INSTRUCTION FOR USE" **REF** Symbol for "CATALOGUE NUMBER"
- EC REP** Symbol for "AUTHORIZED REPRESENTATIVE IN THE EUROPEAN COMMUNITY/EUROPEAN UNION"
-  Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"  Symbol for "KEEP OUT OF THE REACH OF CHILDREN"  Symbol for "MANUFACTURER"
-  Symbol for "TEMPERATURE LIMIT"  Symbol for "KEEP AWAY FROM SUNLIGHT" **MD** Symbol for "MEDICAL DEVICE"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding. For feedback, complaints and reporting of unexpected events contact customer care.