

ViroShield

PREVESTDenPro®
THE FUTURE OF DENTISTRY

Suction Unit Daily Disinfectant

Product Description : ViroShield is a specially formulated, foam free, liquid concentrate for daily cleaning, disinfection and deodorizing medical devices including suction devices and dental unit spittoons.

Composition :

Dimethyldioctylammonium chloride, N-(3-aminopropyl)-N-dodecylpropano-1, 3-Diamine, Quaternary Ammonium Salts, Tetra Potassium Pyrophosphate, IsotridecanolEthoxylated, 2-Aminoethanol, EDTA, Potassium Hydroxide, Dipropylene Glycol Methyl Ether and IPA

Intended Purpose/Intended Use: It is intended to be for cleaning, decontaminating and disinfecting medical device and surfaces.

Indications for use:

It is indicated for daily hygiene and disinfection of dental suction systems as a part of their general disinfecting routine.

Instructions for Use : The concentrate is to be diluted with water to 2% concentration. Pass the solution through the suction system once or twice a day. Leave for 15 minutes to take the effect. After disinfection, rinse the suction system with the potable water.

Warnings : Harmful if swallowed. Causes severe skin burns and eye damage.

IF SWALLOWED: Rinse mouth. Do NOT induce vomiting.

IF ON SKIN(or hair): Take off immediately all contaminated clothing. Rinse skin with water/shower.

IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses if present and continue rinsing. Get medical advice/attention if you feel unwell.

Precautions :

- ♦ Advised for use on surfaces of medical devices by professionals only.
- ♦ Wear personal protective equipment such as gloves/protective clothing/eye protection/ masks for face protection. Wash your hands after handling.
- ♦ Do not mix with other cleaning agents or disinfectants.

Contraindications : Avoid heat

Storage Conditions : Store at temperature between 10°C(50°F) to 24°C(75°F). Keep away from moisture and heat. Close the cap carefully after use. The shelf life of the product is valid only if left unopened and correctly stored in a cool dry place.

Shelf life : 3 years from the date of manufacturing.

Presentation :

Ref: 16001

1 Litre Dosing Bottle

Ref: 16002

5 Litre Can

Disposal Conditions : To be disposed off based on the prevailing rules of the country in which it is used.

Prevest DenPro Limited

Unit II, Export Promotion Industrial Park,
(EPIP), Bari Brahmna, Jammu-181133,

India. www.prevestdenpro.com

Customer Care No. : + 91 8899074151

Email : customercare@prevestdenpro.com

Made in India

CE

Revision :	00
Date :	06-05-2023

UDI Symbol for "UNIQUE DEVICE IDENTIFIER"

LOT Symbol for "BATCH CODE"  Symbol for "USE-BY DATE"  Symbol for "NON STERILE"  Symbol for "KEEP DRY"

 Symbol for "CAUTION"  Symbol for "CONSULT INSTRUCTION FOR USE" **REF** Symbol for "CATALOGUE NUMBER"

EC REP Symbol for "AUTHORIZED REPRESENTATIVE IN THE EUROPEAN COMMUNITY/EUROPEAN UNION"

 Symbol for "DO NOT USE IF PACKAGE IS DAMAGED"  Symbol for "KEEP OUT OF THE REACH OF CHILDREN"  Symbol for "MANUFACTURER"

 Symbol for "TEMPERATURE LIMIT"  Symbol for "KEEP AWAY FROM SUNLIGHT" **MD** Symbol for "MEDICAL DEVICE"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding. For feedback, complaints and reporting of unexpected events contact customer care.