

# ViroShield PDR

**PREVESTDenPro®**  
THE FUTURE OF DENTISTRY

**Powder Concentrate for Suction Unit Disinfection**

**Product Description :** ViroShield PDR is a powder concentrate for weekly hygiene and disinfection of dental suction systems of all types including their drainage pipes and spittoons.

**Composition :**

Sodium Percarbonate, Tetraacetythylenediamine, Citric Acid Anhydrous, Disodium Metasilicate, Alcohol Ethoxylated, Sodium Carbonate, Trisodium Orthophosphate, Enzyme.

**Intended Purpose/Intended Use:** It is intended to be used for cleaning, decontaminating and disinfecting medical device and surfaces.

**Indications for use:**

It is indicated for weekly hygiene and disinfection of dental suction systems as a part of their general disinfecting routine.

**Instructions for Use :** 20g powder to be mixed with 1 liter of luke warm water. Aspirate the solution through the high and low volume suction tubes. Leave for 30minutes. Rinse thoroughly with clean water.

**Warnings :** Harmful if swallowed. Causes severe skin burns and eye damage.

IF SWALLOWED: Rinse mouth. Do NOT induce vomiting.

IF ON SKIN(or hair): Take off immediately all contaminated clothing. Rinse skin with water/shower.

IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses if present and continue rinsing. Get medical advice/attention if you feel unwell.

**Precautions :**

♦ Advised for use on surfaces of medical devices by professionals only.

♦ Wear personal protective equipment such as gloves/protective clothing/eye protection/ masks for face protection. Wash your hands after handling.

♦ Do not mix with other products.

**Contraindications :** Avoid heat

**Storage Conditions :** Store at temperature between 10°C(50°F) to 24°C(75°F). Keep away from moisture and heat. Close the cap carefully after use. The shelf life of the product is valid only if left unopened and correctly stored in a cool dry place.

**Shelf life :** 3 years from the date of manufacturing.

**Presentation :**

Ref : 16003

500g Jar

Ref : 16004

1Kg Jar

**Disposal Conditions :** To be disposed off based on the prevailing rules of the country in which it is used.

**Prevest DenPro Limited**  
Unit II, Export Promotion Industrial Park,  
(EPIP), Bari Brahmna, Jammu-181133,  
India. [www.prevestdenpro.com](http://www.prevestdenpro.com)  
**Customer Care No. :** + 91 8899074151  
**Email :** [customercare@prevestdenpro.com](mailto:customercare@prevestdenpro.com)  
Made in India



<b>Revision :</b>	00
<b>Date :</b>	06-05-2023

Symbol for "UNIQUE DEVICE IDENTIFIER"

Symbol for "BATCH CODE" Symbol for "USE-BY DATE" Symbol for "NON STERILE" Symbol for "KEEP DRY"

Symbol for "CAUTION" Symbol for "CONSULT INSTRUCTION FOR USE" Symbol for "CATALOGUE NUMBER"

Symbol for "AUTHORIZED REPRESENTATIVE IN THE EUROPEAN COMMUNITY/EUROPEAN UNION"

Symbol for "DO NOT USE IF PACKAGE IS DAMAGED" Symbol for "KEEP OUT OF THE REACH OF CHILDREN" Symbol for "MANUFACTURER"

Symbol for "TEMPERATURE LIMIT" Symbol for "KEEP AWAY FROM SUNLIGHT" Symbol for "MEDICAL DEVICE"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding. For feedback, complaints and reporting of unexpected events contact customer care.